

## INSTRUCTIONS FOR BELLY BUTTON BAG PORTS

Item: ST6101

For Single-Use

**Description:** Steri-Tamp® provides assistance in monitoring whether injectable products have already been used by indicating “OPENED” when the metalized layer is removed.

**Intended Purpose:** The Steri-Tamp® purpose is to provide means to identify if a previously opened belly button bag, IV bag port, or 13mm vial top has been tampered with.

**Precautions:** Contents have been GAMMA sterilized and intended for single use. The target area remains sterile until removed from liner. Allied Pharmacy Products does not guarantee or warrant sterility of the medications with which Steri-Tamp® may be used.

**Warning:** Steri-Tamp® is to be used by a trained medical professional. Steri-Tamp® cannot be a substitute for good hospital practice and procedures.

**IMPORTANT:** Allow product to come to room temperature before application (20°C or 68°F).


**Step 1:** Remove Steri-Tamp® from liner.



**Step 2:** Position the registration mark over center of the injection port.



**Step 3:** Press with thumb to firmly affix to port.

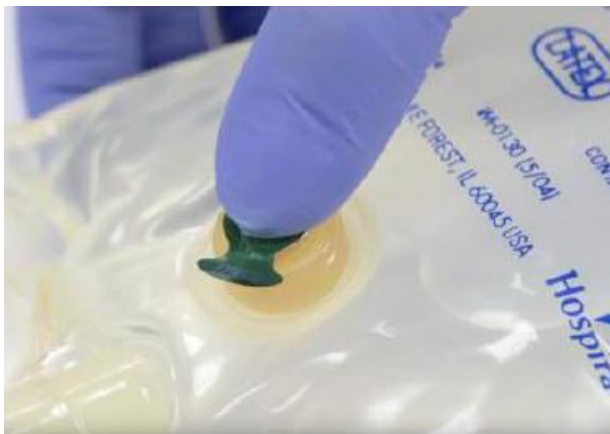

**Step 4:** Removing the top layer leaves behind a printed ‘OPENED’ warning.



\*ST6101 can also be applied to 13mm metal top vials