



[www.steri-tamp.com](http://www.steri-tamp.com)

## INSTRUCTIONS FOR LARGE IV BAG PORTS

Items: ST5112, ST5103 & ST7103

For Single-Use

**Description:** Steri-Tamp® provides assistance in monitoring whether injectable products have already been used by indicating “OPENED” when the metalized layer is removed.

**Intended Purpose:** The Steri-Tamp® purpose is to provide means to identify if a previously opened IV bag port has been tampered with.

**Precautions:** Contents have been GAMMA sterilized and intended for single use. The target area remains sterile until removed from liner. Allied Pharmacy Products does not guarantee or warrant sterility of the medications with which Steri-Tamp® may be used.

**Warning:** Steri-Tamp® is to be used by a trained medical professional. Steri-Tamp® cannot be a substitute for good hospital practice and procedures.

**\*PLEASE BE AWARE, APPLICATION OF STERI-TAMP® IS DIFFERENT THAN OTHER SEALS. FOLLOW THE INSTRUCTIONS BELOW.**

**IMPORTANT:** Allow product to come to room temperature before application (20°C or 68°F).

**Step 1:** Align the back end of the seal with the base of the injection port.



**Step 2:** Fold seal over top of injection port



**Step 3:** Compress and twist around the port squeezing all the air out between the two layers and bonding the seal to the bag port.

**Finished Product:** Impression of injections port should be visible through the seal. Removing the top layer leaves behind an ‘OPENED’ warning.

### REMEMBER TO TWIST!

