

Steri-Tamp®


www.steri-tamp.com

INSTRUCTIONS FOR LARGE IV BAG PORTS

Steri-Tamp® provides assistance in monitoring whether injectable products have already been used by indicating “OPENED” when the metalized layer is removed. Steri-Tamp® cannot be a substitute for good hospital practice and procedures. Allied Pharmacy Products does not guarantee or warrant sterility of the medications with which Steri-Tamp® may be used.

STERILE: Contents have been GAMMA sterilized. The target area remains sterile until removed from liner.


Step 1: Remove Steri-Tamp® from liner.

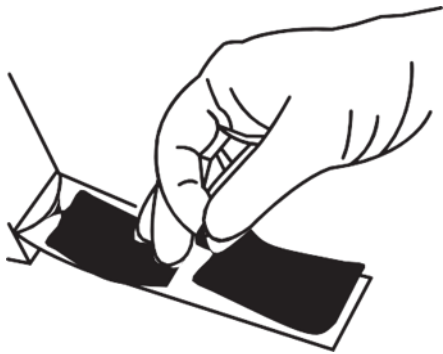

Step 2: Position the registration mark over center of the injection port.



Step 3: Fold seal over top of injection port, compress and twist around the bag port squeezing all the air out between the two layers and bonding to the bag port.

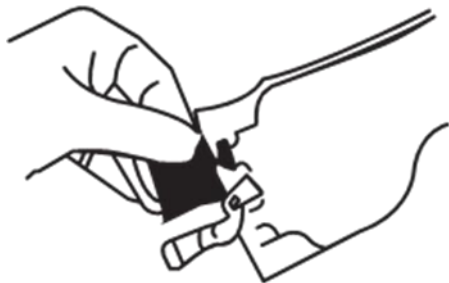

Step 4: Upon removal, the top layer will separate from adhesive strip.



EC REP RMS UK LTD
28 Trinity Road
Nailsea, Somerset
BS48 4NU
England

Custom Tape Company Inver Grove Heights, MN, USA

Steri-Tamp® is a patented and trademark product

CE0297
STERILE R

REVISION A