

Steri-Tamp®

www.steri-tamp.com

INSTRUCTIONS FOR SYRINGES AND CONTAINERS

Steri-Tamp® provides assistance in monitoring whether injectable products have already been used by indicating “OPENED” when the metalized layer is removed. Steri-Tamp® cannot be a substitute for good hospital practice and procedures. Allied Pharmacy Products does not guarantee or warrant sterility of the medications with which Steri-Tamp® may be used.

STERILE: Contents have been GAMMA sterilized. The target area remains sterile until removed from liner.

Step 1: Remove Steri-Tamp® from liner.



Step 2: Position ends of Steri-Tamp® over syringe with the center covering the syringe cap.



Step 3: Press ends firmly to affix.



Step 4: Upon removing the syringe cap, Steri-Tamp® will tear.



Steri-Tamp® is a patented and trademark product

REVISION A