

Steri-Tamp®


www.steri-tamp.com

INSTRUCTIONS FOR VIALS AND BOTTLES

Steri-Tamp® provides assistance in monitoring whether injectable products have already been used by indicating “OPENED” when the metalized layer is removed. Steri-Tamp® cannot be a substitute for good hospital practice and procedures. Allied Pharmacy Products does not guarantee or warrant sterility of the medications with which Steri-Tamp® may be used.

STERILE: Contents have been GAMMA sterilized. The target area remains sterile until removed from liner.

Step 1: Remove Steri-Tamp® from liner.

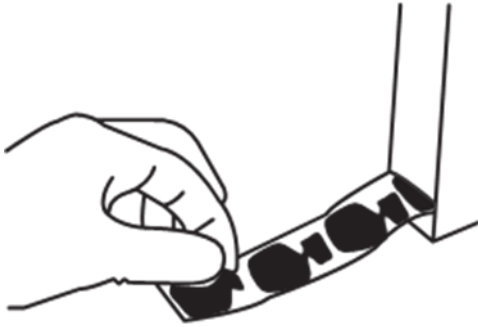

Step 2: Use thumb and index finger to position registration mark over center opening on target area.



Step 3: Press with thumb to firmly affix to top.



Step 4: Upon removal, the top layer will separate from adhesive ring.



EC REP RMS UK LTD
28 Trinity Road
Nailsea, Somerset
BS48 4NU
England

Custom Tape Company Inver Grove Heights, MN, USA

Steri-Tamp® is a patented and trademark product

CE0297
STERILE R

REVISION A