

●Steri-Tamp®

www.steri-tamp.com

INSTRUCTIONS FOR BELLY BUTTON BAG PORTS

>gj 'Kf_d %Mk]


Steri-Tamp® provides assistance in monitoring whether injectable products have already been used by indicating "OPENED" when the metalized layer is removed. Steri-Tamp® cannot be a substitute for good hospital practice and procedures. Allied Pharmacy Products does not guarantee or warrant sterility of the medications with which Steri-Tamp® may be used.

STERILE: Contents have been GAMMA sterilized. The target area remains sterile until removed from liner.

Step 1: Remove Steri-Tamp® from liner.



Step 2: Position the registration mark over center of the injection port.

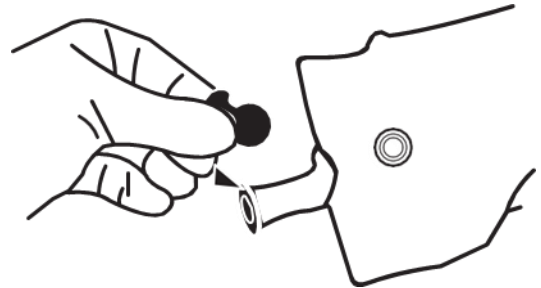

Step 3: Press with thumb to firmly affix to port.



Step 4: Upon removal, the top layer will separate from adhesive ring.



EC REP RMS UK LTD
28 Trinity Road
Nailsea, Somerset
BS48 4NU
England

Curioni Tape Company Inver Grove Heights, MN, USA

CE 0297
STERILE R

Steri-Tamp® is a patented and trademark product

REVISION C