

● Steri-Tamp®

www.steri-tamp.com

INSTRUCTIONS FOR SYRINGES AND CONTAINERS

>gj 'Kā _d %KJ

Steri-Tamp® may provide assistance in monitoring whether injectable products have already been used. When removing the syringe cap, Steri-Tamp® will tear. Steri-Tamp® cannot be a substitute for good hospital practice and procedures. Allied Pharmacy Products does not and can not guarantee or warrant sterility of the medications with which Steri-Tamp® may be used.

Step 1: Remove Steri-Tamp® from liner.



Step 2: Position ends of Steri-Tamp® over syringe with the center covering the syringe cap.



Step 3: Press ends firmly to affix.



Step 4: Upon removing the syringe cap, Steri-Tamp® will tear.

