

# Steri-Tamp®

www.steri-tamp.com

## INSTRUCTIONS FOR LARGE IV BAG PORTS

For Single-Use


Steri-Tamp® provides assistance in monitoring whether injectable products have already been used by indicating “OPENED” when the metalized layer is removed. Steri-Tamp® cannot be a substitute for good hospital practice and procedures. Allied Pharmacy Products does not guarantee or warrant sterility of the medications with which Steri-Tamp® may be used.

STERILE: Contents have been GAMMA sterilized. The target area remains sterile until removed from liner.

**\*PLEASE BE AWARE, APPLICATION OF STERI-TAMP® IS DIFFERENT THAN OTHER SEALS. FOLLOW THE INSTRUCTIONS BELOW.**

**DON'T FORGET TO TWIST!**

**Step 1:** Align the back end of the seal with the base of the injection port.

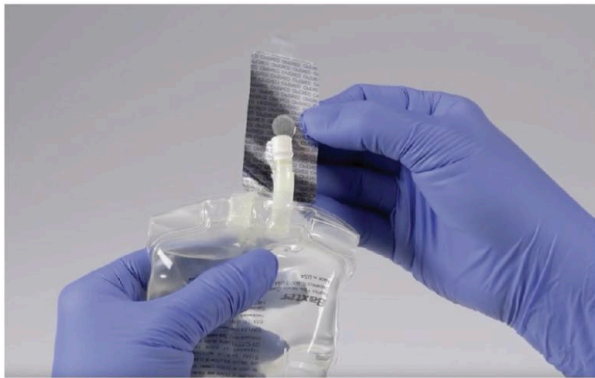

**Step 2:** Fold seal over top of injection port,



**Step 3:**  
Compress and twist around the bag port squeezing all the air out between the two layers and bonding the seal to the bag port.



**Finished Product:**  
Impression of injection port should be visible through the seal. Upon removal, the top layer will separate from the adhesive strip.

