

Steri-Tamp®

www.steri-tamp.com

INSTRUCTIONS FOR LARGE IV BAG PORTS

For Single-Use

Steri-Tamp® provides assistance in monitoring whether injectable products have already been used by indicating “OPENED” when the metalized layer is removed. Steri-Tamp® cannot be a substitute for good hospital practice and procedures. Allied Pharmacy Products does not guarantee or warrant sterility of the medications with which Steri-Tamp® may be used.

STERILE: Contents have been GAMMA sterilized. The target area remains sterile until removed from liner.

***PLEASE BE AWARE, APPLICATION OF STERI-TAMP® IS DIFFERENT THAN OTHER SEALS. FOLLOW THE INSTRUCTIONS BELOW.**

DON'T FORGET TO TWIST!

Step 1: Align the back end of the seal with the base of the injection port.



Step 2: Fold seal over top of injection port,



Step 3:

Compress and twist around the bag port squeezing all the air out between the two layers and bonding the seal to the bag port.



Finished Product:

Impression of injection port should be visible through the seal. Upon removal, the top layer will separate from the adhesive strip.

