

# ●Steri-Tamp®

www.steri-tamp.com

## INSTRUCTIONS FOR BELLY BUTTON BAG PORTS

For Single-Use

Steri-Tamp® provides assistance in monitoring whether injectable products have already been used by indicating “OPENED” when the metalized layer is removed. Steri-Tamp® cannot be a substitute for good hospital practice and procedures. Allied Pharmacy Products does not guarantee or warrant sterility of the medications with which Steri-Tamp® may be used.

STERILE: Contents have been GAMMA sterilized. The target area remains sterile until removed from liner.


**Step 1:** Remove Steri-Tamp® from liner.



**Step 2:** Position the registration mark over center of the injection port.



**Step 3:** Press with thumb to firmly affix to port.

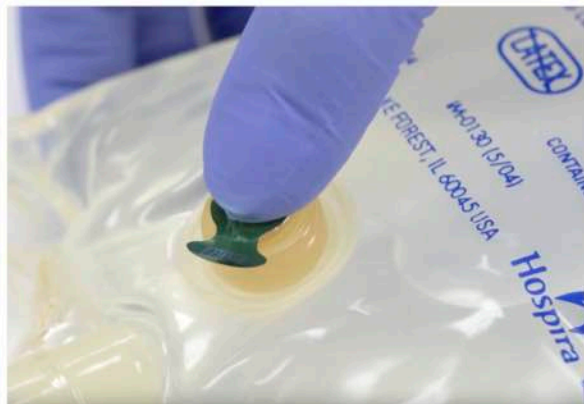

**Step 4:** Upon removal, the top layer will separate from adhesive ring.

