

Steri-Tamp®

www.steri-tamp.com


INSTRUCTIONS FOR VIALS AND BOTTLES

For Single-Use

Steri-Tamp® provides assistance in monitoring whether injectable products have already been used by indicating “OPENED” when the metalized layer is removed. Steri-Tamp® cannot be a substitute for good hospital practice and procedures. Allied Pharmacy Products does not guarantee or warrant sterility of the medications with which Steri-Tamp® may be used.

STERILE: Contents have been GAMMA sterilized. The target area remains sterile until removed from liner.

Step 1: Remove Steri-Tamp® from liner.

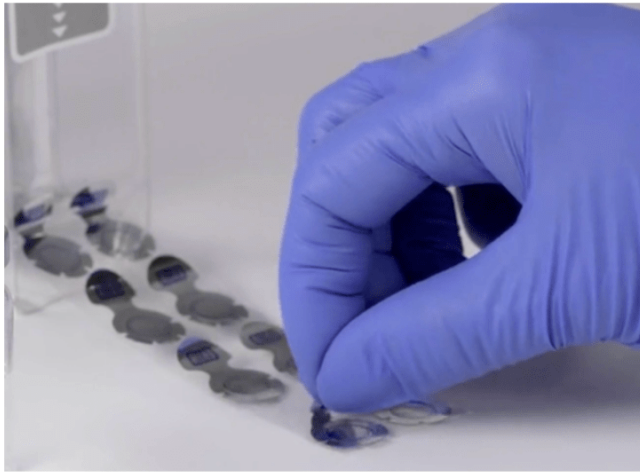

Step 2: Use thumb and index finger to position registration mark over center opening on target area.



Step 3: Press around the seal with finger to firmly affix to top.



Step 4: Upon removal, the top layer will separate from adhesive ring.

